# Symphony<sup>®</sup>

*No. 1 choice  
of hospitals  
& mothers*

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## 9. Troubleshooting

### 9.1 Troubleshooting

Problem	Solution
<b>If the motor is not running</b>	<p>Check whether mains power/battery is available.</p> <p>Check that the device is switched on.</p> <p>Check whether the program card is inserted correctly in the card compartment on the back of the breast pump.</p>
<b>If there is low or no suction</b>	<p>Assemble the pump set in accordance with the instructions in chapter 6.4.</p> <p>Check that all connections of the pump set and breast pump are secure.</p> <p>Express in accordance with chapter 7. Make sure that the breast shield forms a complete seal around the breast.</p> <p>The white valve membrane must lie flat against the yellow valve head.</p> <p>Check that the valve head and membrane are clean and not damaged. See chapter 6.4 "Assembling the pump set".</p> <ul style="list-style-type: none"><li>  The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap.</li><li>  The protective membrane and membrane cap should be correctly assembled and the membrane cap must be firmly fitted to the breast pump membrane to form a seal.</li><li>  The protective membrane must be intact.</li><li>  Do not kink or trap the tubing during expression.</li><li>  All components must be completely dry.</li></ul>
<b>If there is an overflow</b>	<ul style="list-style-type: none"><li>  Place the pump set into the bottle holder.</li><li>  Turn off the pump and unplug the power cord from the socket.</li><li>  Use a damp (not wet) cloth to wipe the pump and membrane area.</li><li>  If there is milk, condensation or visible residues in the tubing: Clean the pump set as described in chapter 5.2.</li><li>  When the components are completely dry, reassemble the breast pump and accessories set.</li></ul>

## 9. Troubleshooting

### 9.2 Error codes

#### Error codes

The following describes error codes that may be displayed indicating a problem with the unit, potential causes, and steps to correct the problem.

	Potential problems	Actions
Invalid card	Program card is not a Symphony card	– Check if the correct program card is being used
	Program card is incorrectly inserted	– Check that card is positioned correctly
	Program card is damaged and not working	– Replace program card
No card	Program card is not inserted	– Insert program card
	Program card is not completely inserted	– Push card in all the way
Battery low	Battery needs charging soon, and will beep every 20 seconds (15 minutes of pumping time remaining from when the indication started)	– Be sure power cord is plugged into the back of the pump
Motor overload	Motor is drawing too much current	– Contact the rental station if this happens repeatedly
Error	Potential fault in electronics of gear control	– Contact the rental station if this happens repeatedly
Power fault	Potential fault in electronics of power supply	– Contact the rental station if this happens repeatedly

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Medela France SAS
Medela Italia Srl a socio unico